

NEW Manual Lock

- The new Manual Lock replaces the pull ring lock that was previously located on the back of the ankle.
- The Manual Lock can be **activated** by double pressing the Change Footwear Button located on the keypad or by activating within the Kinnex App.
- The Change Footwear Button white indicator light will turn on while the ankle is locked.
- To **disengage** the Manual Lock, press the Change Footwear Button twice quickly again or turn off in the App. Once disengaged, the Change Footwear Button light will turn off and the ankle will return to normal function.
- Locks in any position throughout the available ROM.
- **CANNOT** be activated when battery is dead.

NEW Low Battery Feature

Low Battery Mode notifications


- When the battery charge level is at or below 20%, the ankle will periodically vibrate.
- While walking, the vibration feels like a grinding sensation as the user takes a step.
- While not walking, the vibration feels like a subtle buzzing.
- These notifications can be dismissed by quickly pressing and releasing the Battery Button on the front of the ankle, or by using the mobile app.

20% charge indicator: When the battery charge level falls to 20%, the ankle will notify every 15 seconds until dismissed. No indicator lights will flash.

10% charge indicator: When the battery charge level falls to 10%, the indicator light at the front of the ankle will flash red, and the ankle will notify every 5 seconds until dismissed.


2% charge indicator: When the battery charge level falls to 2%, the indicator light at the front of the ankle will flash red, and the ankle will notify every 2 seconds. Notifications cannot be dismissed when the battery charge level is 2% or lower.

0% charge indicator: When the battery charge level falls to 0%, the ankle will power off and go into a low stiffness mode. (Reminder: the Manual Lock cannot be activated when the battery is dead)




Battery / Power Status Button

- Check battery power levels




Change Footwear Button

- Change footwear and calibrate ankle to new shoes
- Manual Lock feature



Bluetooth Button

- Pair ankle to smart device

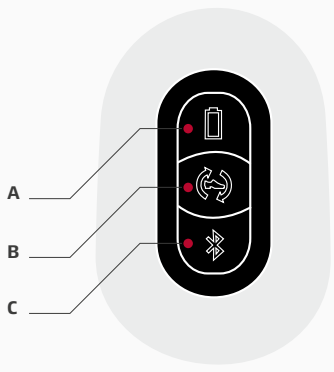

Indicator Light	Charge Level
Solid green	51% – 100%
Flashing green	21% – 50%
Solid red	11% – 20%
Flashing red	1% – 10%

NEW Alignment Setup Feature

- With the new Alignment Setup Feature, you can now lock the ankle in neutral position of 90° with 10mm heel height for establishing alignment and performing sensor calibration.
- To access this feature, tap “Full Setup” in the GaitLab App to get started. Rotate the ankle to the neutral position until it locks into place.
- Note: The Alignment Lock setup is different from the Manual Lock accessible in the Kinnex App. This lock will only lock the ankle in a 90° angle.



For a detailed video guide, watch the What’s New with Kinnex 2.0 video on Youtube: <https://youtu.be/znm9DIMbo2k>

For full product instructions, please reference the product guides and IFU.

We are here for you!

Dear Freedom Innovations customer,

We are available from 8:30 to 17:00 for your questions regarding Kinnex 2.0. Please do not hesitate to give us a call.

Your contact persons in Technical Service:



Technical Manager Europe

Felix Brinckmann

Mobile: +49 - (0) 151 645 05 369

Email: fbrinckmann@freedom-innovations.eu



Clinical Trainer

Christian Bieber

Mobile: +49 - (0) 177 485 55 93

Email: cbieber@freedom-innovations.eu

Your contact persons in Service:

- ▶ Phone: +31 - (0) 53-20 30 300
- ▶ Info@freedom-innovations.eu

