



FITTING INSTRUCTIONS

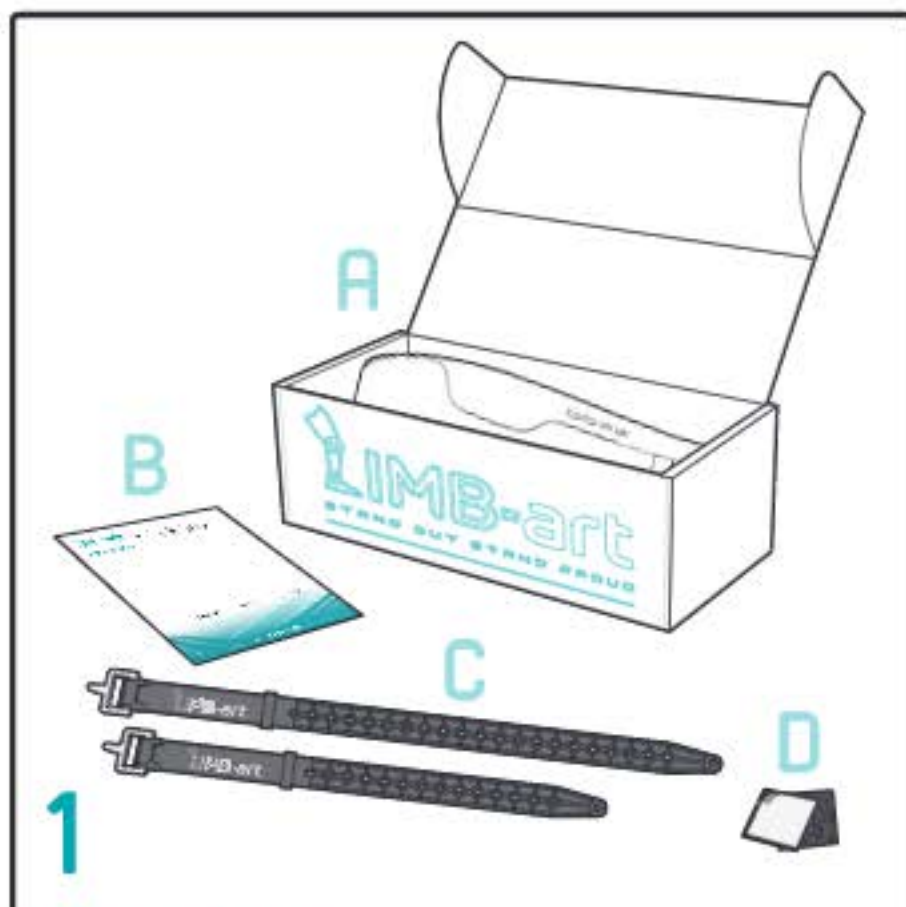


Pack contents:


- A. LIMB-art cover
- B. Fitting instructions
- C. Upper strap & Lower strap
- D. Optional self-adhesive Velcro®

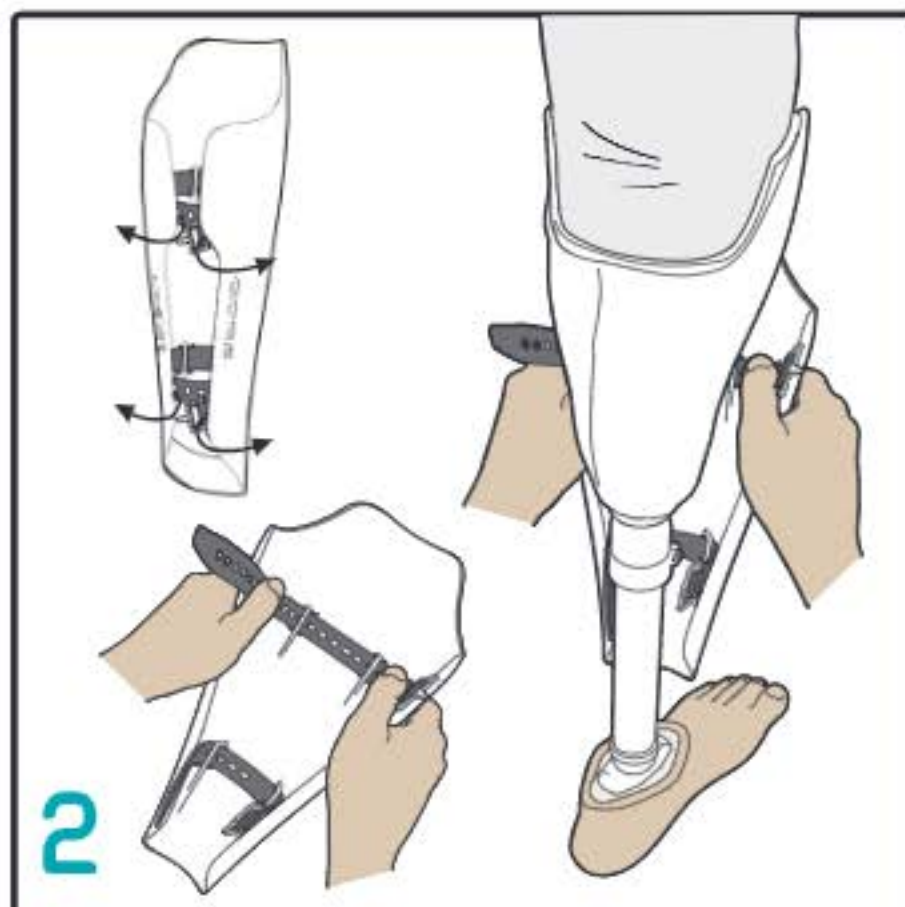

Straps are pre-fitted to your cover. Open both straps, flex open the cover and place over your prosthetic. Start with the bottom strap first.

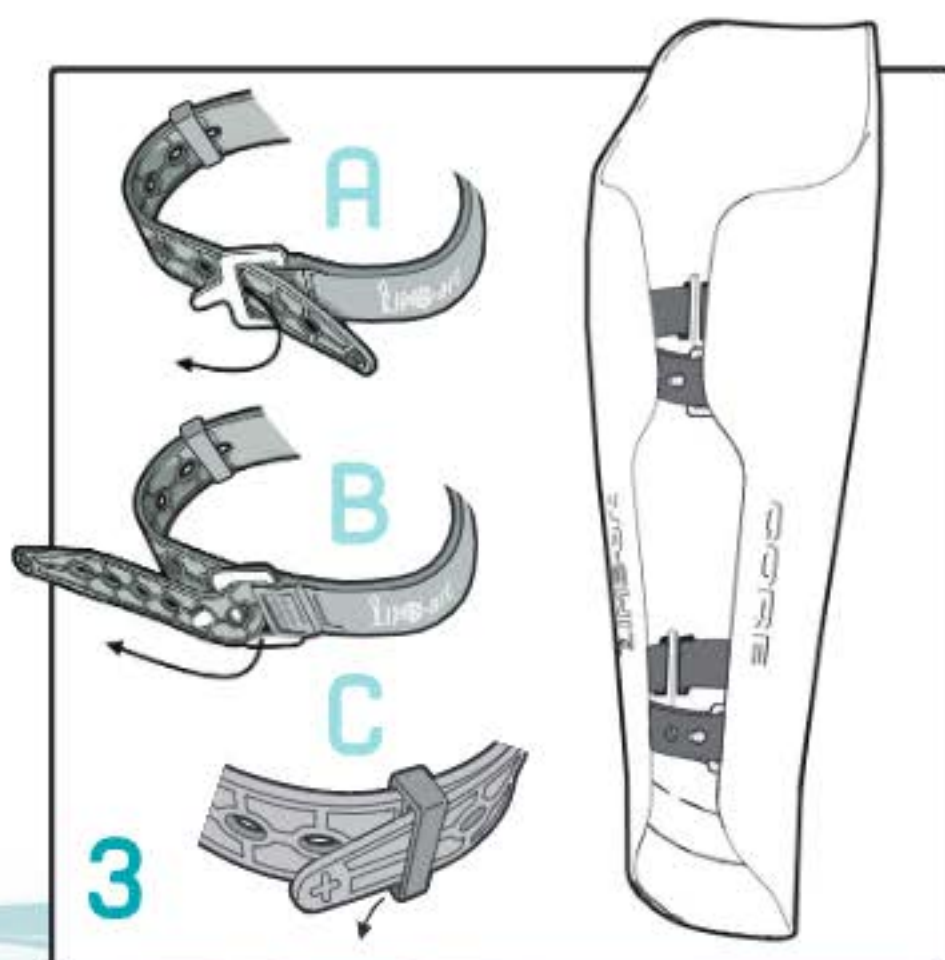

- A. Feed straps through buckles.
- B. Pull tight, then pull strap back over buckle to engage locating hole.
- C. Tuck excess strap through tidy loop or trim if desired.

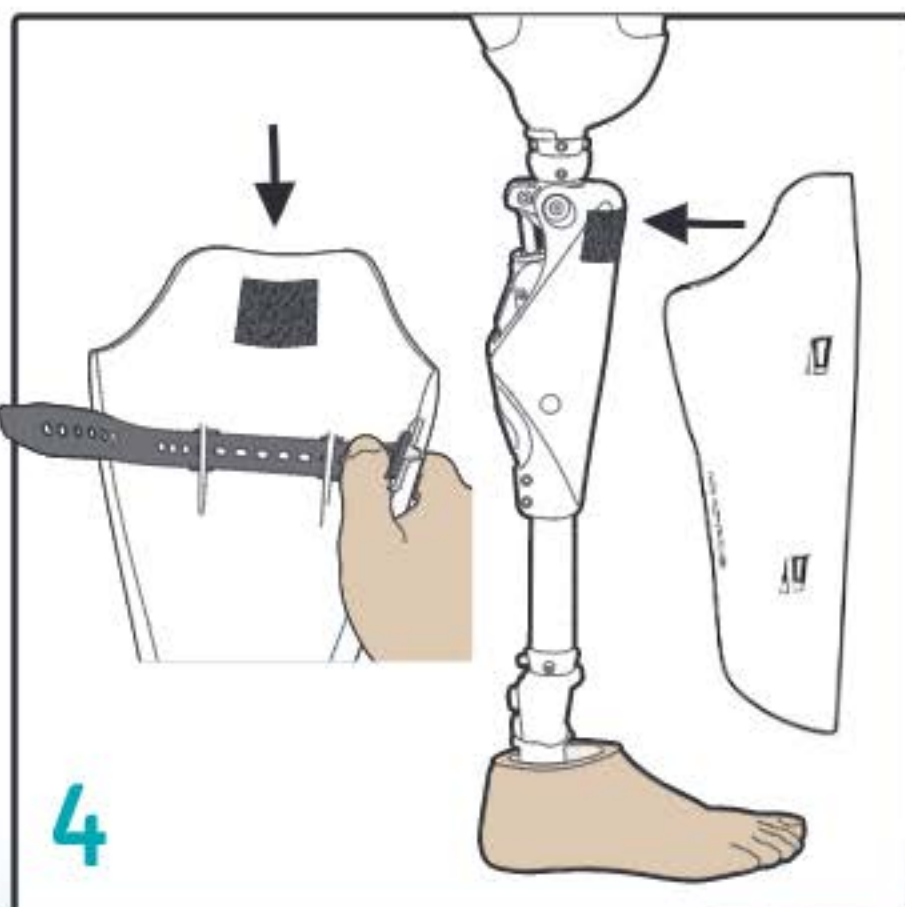

As an alternative to using upper strap, simply affix one side of Velcro® to inside top cover and the other to outside top of prosthetic as shown. (Especially useful for MPK's)