

# APC Torque Setting Chart

Part Number	Description	Bolts/Screw	Torque - ft lbs	Torque - Nm
TITANIUM COMPONENTS				
TI-00MS	Titanium Rotatable Pyramid w/ Male Thread and Center Hole			
TI-01MS	Titanium Rotatable Pyramid w/ Female Thread & Clamp	M5X0.8X25 CLAMP SCREW	7	9
TI-100C	Titanium 30mm Tube Clamp Adapter	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-100C-FL	Titanium 30mm Tube Clamp Adapter	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-100C-LONG	Titanium 30mm Tube Clamp Adapter LONG (84mm)	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-100C-SH	Titanium 30mm Tube Clamp Adapter SHORT w/ Inside KNURL	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-105C	Titanium 30mm Tube Clamp Adapter w/ 5mm Offset Receiver	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-10TC	10" 30mm Pylon w/ Titanium Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-10TC-I	10" 30mm Pylon w/ Internal Titanium Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-110AT	Titanium 30mm Pylon (LONG) w/ Receiver Adapter - Overall Length = 3.8"	M8X1.25X12 SET SCREW	11	15
TI-110SH	Titanium 30mm Pylon (SHORT) w/ Receiver Adapter - Overall Length of 2.06" Accommodates 1 - TI-100C	M8X1.25X12 SET SCREW	11	15
TI-112-P	Titanium 30mm Pylon (SHORT) w/ Pyramid Adapter - Overall Length of 2.46" Accommodates 1 - TI-100C			
TI-115DM	Titanium Dual Pyramid Adapter			
TI-120P	Titanium 30mm Tube Clamp w/ Pyramid Adapter	M5X0.8X20 CLAMP SCREW	7	9
TI-125P	Titanium Pyramid / Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-132D	Titanium 32mm Dual Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-145D	Titanium 45mm Dual Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-160D	Titanium 60mm Dual Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-175D	Titanium 75mm Dual Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-17TC	17" 30mm Pylon w/ Titanium Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-17TC-I	17" 30mm Pylon w/ Internal Titanium Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-205C	Titanium 5 Degree Angled Tube Clamp Adapter	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-210C	Titanium 10 Degree Angled Tube Clamp Adapter	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-215C	Titanium 15 Degree Angled Tube Clamp Adapter	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-220C	Titanium 20 Degree Angled Tube Clamp Adapter	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-400A	Titanium Offset 4 Hole Pyramid Adapter (Axial)			
TI-400B	Titanium 4 Hole Pyramid Adapter			
TI-400BIL	Titanium 4 Hole Pyramid Adapter (ROUND Base)			
TI-400D	Titanium Offset 4 Hole Pyramid Adapter (Diagonal)			
TI-400RM	Titanium 4 Hole Rotatable Pyramid Adapter			

Part Number	Description	Bolts/Screw	Torque - ft lbs	Torque - Nm
TI-406PAT	Titanium Rotatable Pyramid Adapter w/ Threaded 4 Hole Base	M6X20 MOUNTING SCREWS	11	15
		M6X25 MOUNTING SCREWS	11	15
TI-420L	Titanium Round 4 Hole Threaded Plate (.188" Thick)	M6X20 MOUNTING SCREWS	11	15
		M6X25 MOUNTING SCREWS	11	15
TI-425BH	Titanium 4 Hole Pyramid Adapter w/ Center Hole			
TI-450BF	Titanium 4 Hole Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-455RF	Titanium 4 Hole Rotatable Receiver Adapter	M8X1.25X12 SET SCREW	11	15
TI-456PAT	Titanium Rotatable Receiver Adapter w/ 4 Hole Threaded Base	M8X1.25X12 SET SCREW	11	15
		M6X20 MOUNTING SCREWS	11	15
		M6X25 MOUNTING SCREWS	11	15
TI-460B	Titanium 4 Hole Clamping Adapter w/ Center Thread (BASE ONLY)	M5X.08X25 CLAMP SCREW	7	9
		M6X25 MOUNTING SCREWS	11	15
		M6X25 SOCKET HEAD MOUNTING SCREWS	11	15
TI-460TB	Titanium 4 Hole Clamping Adapter w/ Center Thread (BASE ONLY) All 4 Holes Counter Sunk	M5X.08X25 CLAMP SCREW	7	9
		M6X25 MOUNTING SCREWS	11	15
TI-460TRM	Titanium 4 Hole Clamping Adapter (Center Thread) w/ Rotatable Pyramid - All 4 Holes Counter Sunk	M5X0.8X25 CLAMP SCREW	7	9
		M6X25 MOUNTING SCREWS	11	15
TI-465RF	Titanium 4 Hole Clamping Adapter (Center Thread) w/ Rotatable Receiver	M5X0.8X25 CLAMP SCREW	7	9
		M6X25 MOUNTING SCREWS	11	15
		M6X25 SOCKET HEAD MOUNTING SCREWS	11	15
TI-465TRF	Titanium 4 Hole Clamping Adapter (Center Thread) w/ Rotatable Receiver - All 4 Holes Counter Sunk	M5X.08X25 CLAMP SCREW	7	9
		M6X25 MOUNTING SCREWS	11	15
		M8X1.25X12 SET SCREW	11	15
TI-4PRWH	Titanium 4 Hole Pyramid with Center Hole (Round Base w/ Rotation Slots)	M6X1.0X20 CAP HEAD MOUNTING SCREW	11	15
		M6X1.0X25 CAP HEAD MOUNTING SCREW	11	15
TI-4RR	Titanium 4 Hole Receiver (Round Base w/ Rotation Slots)	M6X1.0X20 CAP HEAD MOUNTING SCREW	11	15
		M6X1.0X25 CAP HEAD MOUNTING SCREW	11	15
TI-500P	Titanium Foot Pyramid Adapter (Bolt and Pins Included)	M10X1.25X65 FOOT BOLT	20	32
TI-50FS	Titanium Rotatable Receiver w/ Male Thread	M5X.08X25 CLAMP SCREW	7	9
TI-600AD	Titanium 4 Hole Alignable Pyramid Adapter	M8X1.25X12 SET SCREW	11	15
TI-600-SLA-PM	Titanium (Sleeve System) Pyramid w/ Male Dovetail Slide Adapter	M6x1.0x30 CAP SCREW	11	15
TI-60FS	Titanium Receiver w/ Female Thread	M5X.08X25 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-610-SLA-TF	Titanium (Sleeve System) Male Thread w/ Female Dovetail Slide Adapter	M8X1.25X12 SET SCREW	11	15
TI-615-SLA-CF	Titanium (Sleeve System) 30mm Tube Clamp w/ Female Dovetail Slide Adapter	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-620-SLA-FF	Titanium (Sleeve System) Dual Female Dovetail Slide Adapter	M8X1.25X12 SET SCREW	11	15
TI-630-SLA-RF	Titanium (Sleeve System) Receiver w/ Female Dovetail Slide Adapter	M8X1.25X12 SET SCREW	11	15
TI-640-SLA-PF	Titanium (Sleeve System) Pyramid w/ Female Dovetail Slide Adapter	M8X1.25X12 SET SCREW	11	15
TI-650-SLA-RM	Titanium (Sleeve System) Receiver w/ Male Dovetail Slide Adapter	M8X1.25X12 SET SCREW	11	15
		M6x16 FLAT HEAD	11	15
TI-655-SLA-CM	Titanium (Sleeve System) 30mm Tube Clamp w/ Male Dovetail Slide Adapter	M5X0.8X20 CLAMP SCREW	7	9
		M6x1.0x16 CAP SCREW	7	9
TI-675AR	Titanium Rotatable and Alignable Dual Receiver Adapter (7.5mm Adjustment)	M8X1.25X12 SET SCREW	11	15

Part Number	Description	Bolts/Screw	Torque - ft lbs	Torque - Nm
		M8X1.25X16 SET SCREW	11	15
		M8X1.25X20 SET SCREW	11	15
		M8X1.25X25 SET SCREW	11	15
TI-685AR	Titanium 4 Hole Alignable & Rotatable Receiver Adapter	M8X1.25X12 SET SCREW	11	15
		M8X1.25X20 SET SCREW	11	15
		M8X1.25X25 SET SCREW	11	15
TI-700LM	Stainless Steel 3 Prong Laminating Base (MALE THREAD) w/ Titanium Rotatable Pyramid Adapter	M5X.08X25 CLAMP SCREW	7	9
TI-740-B	Titanium 4 Prong Laminating Base (MALE THREAD)			
TI-740L	Titanium 4 Prong Laminating Base w/ Fixed Pyramid Adapter			
TI-740L-S	Titanium 4 Prong Laminating Adapter (SHORT)			
TI-740-RM	Titanium 4 Prong Laminating Base (MALE THREAD) w/ Titanium Rotatable Pyramid Adapter	M5X.08X25 CLAMP SCREW	7	9
TI-745-RF	Titanium 4 Prong Laminating Base (MALE THREAD) w/ Titanium Rotatable Receiver Adapter	M5X.08X25 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-750LM	Stainless Steel 3 Prong Laminating Base (MALE THREAD) w/ Titanium Rotatable Receiver Adapter	M5X.08X25 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
TI-800LM	Stainless Steel 3 Prong Laminating Base (Female Thread) w/ Titanium Rotatable Pyramid Adapter (Includes	M6X1.0X20 CLAMP SCREW	8.5	12
TI-840LM	Stainless Steel 4 Prong Laminating Base (Female Thread) w/ Titanium Rotatable Pyramid Adapter (Includes	M6X1.0X20 CLAMP SCREW	8.5	12
TI-845LM	Stainless Steel 4 Prong Laminating Base (Female Thread) w/ Titanium Rotatable Receiver Adapter (Includes	M6X1.0X20 CLAMP SCREW	8.5	12
		M8X1.25X12 SET SCREW	11	15
TI-850LM	Stainless Steel 3 Prong Laminating Base (Female Thread) w/ Titanium Rotatable Receiver Adapter (Includes	M6X1.0X20 CLAMP SCREW	8.5	12
		M8X1.25X12 SET SCREW	11	15
TI-CPF-400	Titanium 4 Hole Threaded Laminating Ring (Includes Lamination Cap)	M4-0.7x16 SET SCREW	11	15
		M6X20 MOUNTING SCREWS	11	15
TI-CPF-800	Titanium Laminating Pyramid Adapter (Includes Lamination Cap)			
TI-TF900SET	Titanium Thermoplastic Socket Attachment Plate Set	M6X20 MOUNTING SCREWS	11	15
<b>STAINLESS STEEL COMPONENTS</b>				
SS-00MS	Stainless Steel Rotatable Pyramid w/ Male Thread and Center Hole			
SS-01MS	Stainless Steel Rotatable Pyramid w/ Female Thread & Clamp	M5X.08X25 CLAMP SCREW	7	9
SS-100C	Stainless Steel 30mm Tube Clamp Adapter	M5X0.8X20 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
SS-10TC	10" 30mm Pylon w/ Stainless Steel Receiver Adapter	M8X1.25X12 SET SCREW	11	15
SS-132D	Stainless Steel 32mm Dual Receiver Adapter	M8X1.25X12 SET SCREW	11	15
SS-145D	Stainless Steel 45mm Dual Receiver Adapter	M8X1.25X12 SET SCREW	11	15
SS-160D	Stainless Steel 60mm Dual Receiver Adapter	M8X1.25X12 SET SCREW	11	15
SS-175D	Stainless Steel 75mm Dual Receiver Adapter	M8X1.25X12 SET SCREW	11	15
SS-17TC	17" 30mm Pylon w/ Stainless Steel Receiver Adapter	M8X1.25X12 SET SCREW	11	15
SS-300LM	Stainless Steel 3 Prong Laminating Base (MALE THREAD)			
SS-400B	Stainless Steel 4 Hole Pyramid Adapter			
SS-425BH	Stainless Steel 4 Hole Pyramid Adapter w/Center Hole			
SS-460TB	Stainless Steel 4 Hole Clamping Adapter w/ Female Thread (BASE ONLY)	M6x1.0x20 CLAMP SCREW	7	9
		M6X25 MOUNTING SCREWS	11	15
SS-500P	Stainless Steel Foot Pyramid Adapter (Bolt and Pins Included)	M10X1.25X65 FOOT BOLT	20	32
SS-50FS	Stainless Steel Rotatable Receiver w/ Male Thread	M5X.08X25 CLAMP SCREW	7	9

Part Number	Description	Bolts/Screw	Torque - ft lbs	Torque - Nm
SS-60FS	Stainless Steel Rotatable Receiver w/ Female Thread & Clamp	M5X.08X25 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
SS-700LM	Stainless Steel 3 Prong Laminating Base (MALE THREAD) w/ Stainless Steel Rotatable Pyramid Adapter	M5X.08X25 CLAMP SCREW	7	9
SS-740L	Stainless Steel 4 Prong Laminating Base w/ Fixed Pyramid Adapter			
SS-750LM	Stainless Steel 3 Prong Laminating Base (MALE THREAD) w/ Stainless Steel Rotatable Receiver Adapter	M5X.08X25 CLAMP SCREW	7	9
		M8X1.25X12 SET SCREW	11	15
SS-800BASE	Stainless Steel 3 Prong Laminating Base w/ Female Thread (Includes Lamination Cap)	M6x1.0x20 CLAMP SCREW	7	9
SS-800LM	Stainless Steel 3 Prong Laminating Base (Female Thread) w/ Stainless Steel Rotatable Pyramid Adapter (Includes Stainless Steel 3 Prong Laminating Base w/ Fixed Stainless Steel Pyramid Adapter (Includes Lamination Cap)	M6x1.0x20 CLAMP SCREW	7	9
SS-810LM	Stainless Steel 4 Prong Laminating Base w/ Female Thread (Includes Lamination Cap)			
SS-840BASE	Stainless Steel 4 Prong Laminating Base (Female Thread) w/ Stainless Steel Rotatable Pyramid Adapter (Includes Stainless Steel 4 Prong Laminating Base (Female Thread) w/ Stainless Steel Rotatable Receiver Adapter (Includes	M6X1.0X20 CLAMP SCREW	8.5	12
SS-840LM	Stainless Steel 4 Prong Laminating Base (Female Thread) w/ Stainless Steel Rotatable Pyramid Adapter (Includes	M6X1.0X20 CLAMP SCREW	8.5	12
SS-845LM	Stainless Steel 4 Prong Laminating Base (Female Thread) w/ Stainless Steel Rotatable Receiver Adapter (Includes	M6X1.0X20 CLAMP SCREW	8.5	12
		M8X1.25X12 SET SCREW	11	15
SS-850LM	Stainless Steel 3 Prong Laminating Base (Female Thread) w/ Stainless Steel Rotatable Receiver Adapter (Includes	M6X1.0X20 CLAMP SCREW	8.5	12
		M8X1.25X12 SET SCREW	11	15

## HEAVY DUTY COMPONENTS

HD-100C	Heavy Duty Titanium 34mm Tube Clamp (Rated to 500 lbs)	M6X1.0X20 CLAMP SCREW	8.5	12
		M10X16 SET SCREW	11	15
HD-100C-A	Heavy Duty Titanium 34mm Tube Clamp w/30mm Receiver Adapter (Rated to 500 lbs)	M6X1.0X20 CLAMP SCREW	8.5	12
		M10X16 SET SCREW	11	15
HD-105C	Heavy Duty Titanium 34mm Tube Clamp w/5mm Offset (Rated to 500 lbs)	M6X1.0X20 CLAMP SCREW	8.5	12
		M10X16 SET SCREW	11	15
HD-110AT	Heavy Duty Titanium 34mm Pylon w/ Receiver Adapter (Rated to 500 lbs)	M10X16 SET SCREW	11	15
HD-110AT-LONG	Heavy Duty Titanium 34mm Pylon (LONG) w/ Receiver Adapter - LONG (PYLON = 2.00" - 3.21" Overall) (Rated to Heavy Duty Titanium 34mm Dual Receiver Adapter (Rated to 500 lbs)	M10X16 SET SCREW	11	15
HD-132D	Heavy Duty Titanium 34mm Pyramid Adapter (Rated to 500 lbs)	M10X16 SET SCREW	11	15
HD-425BH	Heavy Duty Titanium 34mm Foot Pyramid Adapter (Rated to 500 lbs)	M6X1.0X20 CLAMP SCREW	8.5	12
HD-500P	Heavy Duty Titanium 34mm Foot Pyramid Adapter w/ Bolt (Rated to 500 lbs)	M14X1.5X70 FOOT BOLT	20	32
HD-740L	Heavy Duty Titanium 34mm 4 Prong Laminating Pyramid (Rated to 500 lbs)			
HD-745L	Heavy Duty Titanium 34mm 4 Prong Laminating Receiver (Rated to 500 lbs)	M10X16 SET SCREW	11	15
HD-745L-A	Heavy Duty Titanium 4 Prong w/30mm Receiver (Rated to 500 lbs)	M10X16 SET SCREW	11	15
HD-AL-PYLON	Heavy Duty 34mm Aluminum Pylon 11.75" Length (Rated to 500 lbs)			

## ALUMINUM COMPONENTS

AL-410-FLPL	Aluminum 10 Degree Flexion Plate	M6x14 MOUNTING SCREW	11	15
AL-415-FLPL	Aluminum 15 Degree Flexion Plate	M6x14 MOUNTING SCREW	11	15
AL-413-OS	Aluminum 4 Hole 13mm Offset Plate	M6x12 MOUNTING SCREW	11	15
		M6x16 MOUNTING SCREW	11	15
AL-419-OS	Aluminum 4 Hole 19mm Offset Plate	M6x12 MOUNTING SCREW	11	15
		M6x16 MOUNTING SCREW	11	15
AL-4X4-SLIDE	Aluminum 4 Hole 25.4mm Slide (Bolts Included)	M6x1.0x16 CAP HEAD MOUNTING SCREW	11	15
		M6x16 MOUNTING SCREW	11	15
AL-425-OS	Aluminum 4 Hole 25mm Offset Plate	M6x12 MOUNTING SCREW	11	15
		M6x16 MOUNTING SCREW	11	15

Part Number	Description	Bolts/Screw	Torque - ft lbs	Torque - Nm
AL-409-OS	Aluminum 4 Hole 9mm Offset Plate	M6x12 MOUNTING SCREW	11	15
		M6x16 MOUNTING SCREW	11	15
AL-420L-SL	Aluminum 4 Hole Laminating Plate with Centered 6 Hole Shuttle Lock Relief	M6x20 MOUNTING SCREW	11	15
		M6x25 MOUNTING SCREW	11	15
ALH-425BH	Aluminum 4 Hole Pyramid Adapter w/Center Hole (250 lb Rating)			
AL-405-FLPL	Aluminum 5 Degree Flexion Plate	M6x14 MOUNTING SCREW	11	15
AL-408-FLPL	Aluminum 8 Degree Flexion Plate	M6x14 MOUNTING SCREW	11	15
ALH-500P	Aluminum Foot Pyramid Adapter (250 lb Rating)	M10X1.25X65 FOOT BOLT	20	32
AL-CPF-400S	Aluminum Pre-form 4 Hole Plate For use with IRS Shuttle Lock			
AL-CPF-400	Aluminum Round 4 Hole Laminating Plate Anodized BLACK (5/8" Thick with 1/2" Tie Groove) Includes Screws	M6x20 MOUNTING SCREW	11	15
		M6x25 MOUNTING SCREW	11	15
AL-420L-OS	Aluminum Round 4 Hole Laminating Plate w/offset Center & 6 Hole Shuttle Lock Relief	M6x20 MOUNTING SCREW	11	15
		M6x25 MOUNTING SCREW	11	15
AL-420L-TH	Aluminum Round 4 Hole Laminating Plate with Threaded Center Hole (3/4 x 16)	M6x20 MOUNTING SCREW	11	15
		M6x25 MOUNTING SCREW	11	15
AL-418B	Aluminum Round 4 Hole Spacer Plate (1/8")			
AL-440L	Aluminum Round 4 Hole Threaded Laminating Ring (3/8" Thick with 1/8" Tie Groove)	M6x20 MOUNTING SCREW	11	15
		M6x25 MOUNTING SCREW	11	15
AL-420L	Aluminum Round 4 Hole Threaded Lamination Plate	M6x20 MOUNTING SCREW	11	15
		M6x25 MOUNTING SCREW	11	15
AL-431L	Aluminum Round 4 Hole Threaded Lamination Plate (.313)	M6x20 MOUNTING SCREW	11	15
		M6x25 MOUNTING SCREW	11	15
AL-416B	Aluminum Square 4 Hole Spacer Plate (.08)			
AL-420B	Aluminum Square 4 Hole Spacer Plate (.134)			
AL-430L	Aluminum Square 4 Hole Threaded Lamination Plate	M6x20 MOUNTING SCREW	11	15
		M6x25 MOUNTING SCREW	11	15
AL-TF900SET	Aluminum Thermoplastic Socket Attachment Plate Set	M6x20 MOUNTING SCREW	11	15
		M6x25 MOUNTING SCREW	11	15

## PEDIATRIC/GERIATRIC COMPONENTS

JR-100C	Pediatric / Geriatric Titanium 22mm Tube Clamp Adapter	M6x10 PEDIATRIC SET SCREWS	8	10
		M4X.7X16 PEDIATRIC CLAMP SCREW	5	7
JR-100C-A	Pediatric / Geriatric Titanium 22mm Tube Clamp Adapter w/30mm Receiver	M8X1.25X12 SET SCREW	11	15
		M4X.7X16 PEDIATRIC CLAMP SCREW	5	7
JR-100C-LONG	Pediatric / Geriatric Titanium 22mm Tube Clamp Adapter LONG (2 5/8" Overall Height)	M6x10 PEDIATRIC SET SCREWS	8	10
		M4X.7X16 PEDIATRIC CLAMP SCREW	5	7
JR-100C-LONG-A	Pediatric / Geriatric Titanium 22mm Tube Clamp Adapter LONG w/30mm Adult Receiver (2 5/8" Overall Height)	M8X1.25X12 SET SCREW	11	15
		M4X0.7X16 PEDIATRIC CLAMP SCREW	5	7
JR-10T	Pediatric / Geriatric 10" 22mm Pylon			
JR-10TC	Pediatric / Geriatric 10" 22mm Pylon w/Titanium Receiver	M6x10 PEDIATRIC SET SCREWS	8	10
JR-10TC-A	Pediatric / Geriatric 10" 22mm Pylon w/30mm Adult Titanium Receiver	M8X1.25X12 SET SCREW	11	15
JR-10TC-I	Pediatric / Geriatric 10" 22mm Pylon w/Internal Titanium Receiver	M6x10 PEDIATRIC SET SCREWS	8	10
JR-400A	Pediatric / Geriatric Titanium 4 Hole Offset Pyramid (AXIAL)			
JR-400B	Pediatric / Geriatric Titanium 4 Hole Pyramid			

Part Number	Description	Bolts/Screw	Torque - ft lbs	Torque - Nm
JR-400D	Pediatric / Geriatric Titanium 4 Hole Offset Pyramid (DIAGONAL)			
JR-416B	Pediatric / Geriatric Aluminum 4 Hole Spacer Plate (1/16" Thick)			
JR-425BH	Pediatric / Geriatric Titanium 4 Hole Pyramid w/Center Hole			
JR-450BF	Pediatric / Geriatric Titanium 4 Hole Receiver Adapter	M6x10 PEDIATRIC SET SCREWS	8	10
JR-500P	Pediatric / Geriatric Foot Pyramid Adapter (Includes Bolt)	M8-1.25x50 PEDIATRIC FOOT BOLT	15	21
JR-AL-417B	Pediatric / Geriatric Aluminum 4 Hole Spacer (.134 Thick)			
JR-AL-420L	Pediatric / Geriatric Aluminum Round 4 Hole Threaded Plate (M6 Screws Included)	M6x20 MOUNTING SCREW	11	15
		M6x25 MOUNTING SCREW	11	15
JR-AL-420L-M5	Pediatric / Geriatric Aluminum Round 4 Hole Threaded Plate (M5 Screws Included)	M5x20 PEDIATRIC MOUNTING SCREW	11	15
JR-AL-431L-M5	Pediatric / Geriatric Round 4 Hole Threaded Plate .313 Thick (M5 Screws Included)	M5x20 PEDIATRIC MOUNTING SCREW	11	15
JR-AL-4X4-SLIDE	Pediatric / Geriatric Aluminum 4 Hole 18mm Slide (M5 Screws Included)	M5x14 PEDIATRIC MOUNTING SCREW	11	15
		M5x20 PEDIATRIC MOUNTING SCREW	11	15